





U.S. Patent No. 8,844,193.B2

- ARCTICK is a non-toxic spray that rapidly freezes and kills ticks before removal
- Using ARCTICK to freeze ticks fights the transfer of diseases
- It is environmentally friendly and safe
- Fights tramsmission of Diseases; **Effectively helps prevent Lyme Disease**
- "Freeze it; Don't squeeze it!", the ASCIA* recommended method
- Reduces the risk of tick head embedding in skin.
- Reduces the risk of injection of toxic bacteria into the body.

NEW TECHNOLOGY & DESIGN!

- Precision FREEZE APPLICATOR
- Freezes and kills ticks rapidly
- Canister contains up to 25 Doses
- Metered dose, prevents frostbites
- Refill canisters are available



- To easily remove frozen-dead tick without damage
- Unique design for lifting ticks without the squeeze
- A safe alternative to pesticide treatment
- NO PESTICIDES, OZONE FRIENDLY
- Contains no CFCs or other ozone depleting substances
- Safe when used as directed
- Laboratory tested





DIRECTIONS FOR USE:

- 1. Locate the tick.
- 2. Place the applicator cage over the tick. Cage must be facing down.
- 3. Press top of applicator with thumb until spray stream stops. Spray once, a second time for larger ticks. Do not exceed three (3) sprays per area.
- 4. Place extractor supplied under the whole tick and pull straight up.
- 5. Wash area with soap and water.





Spectrum Catalysts, Inc. 202 Christopher Street Ronkonkoma, NY 11779

Learn more about ARCTICK® by visiting www.ARCTICKSPRAY.com

www.SpectrumCatalyst.com